


Package Insert for Personal Use Kit


Carefully follow these instructions to make sure you get accurate test results.

Stool Sample Collection and Submission

Step 1 Fill in required information on sampling bottle.
(Name, Date Of Birth, and Sample Collect Date)

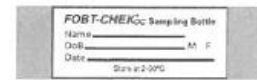

Step 2 Unfold and place collection paper inside toilet bowl on top of water. Make sure sampling bottle is nearby.

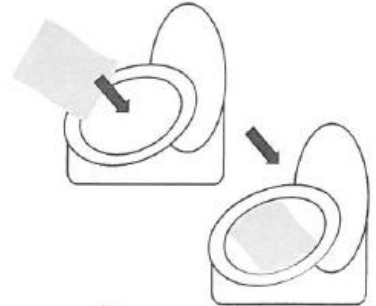


Step 3 Deposit stool sample on top of collection paper. Do not urinate on the paper.

Step 4 Open green cap of sampling bottle by twisting and lifting.




Step 5 Immediately scrape the surface of the stool with the sample probe. Cover grooved portion of probe completely with stool. Sample must be collected before paper sinks and sample touches water.

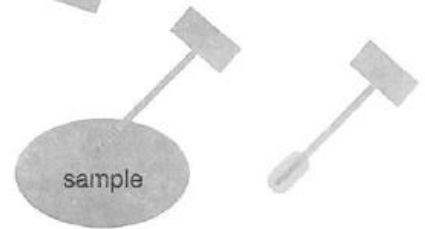

Step 6 Insert probe into bottle and snap green cap on tightly. Do not reopen.



Step 7 Wrap bottle in absorbant pad and place in biohazard bag. Put the biohazard bag containing the specimen in the mailing envelope provided.

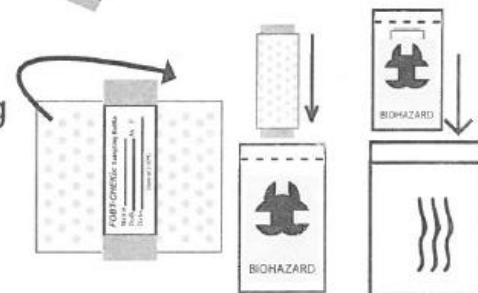

Step 8 Return the envelope to your laboratory
(Per the instruction label on the kit envelope)

Step 9 Flush collection paper. It is biodegradable and will not harm septic systems.